## **Extender** User manual

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The Extender is an additional aid for putting on Therapeutic Elastic Stockings (TES), also known as support stockings. You use the Extender together with the electrical HelpSoq On device.

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[www.helpsoq.com](http://www.helpsoq.com)

## Extender - User Manual

Read the safety instructions carefully and in full, before taking the Extender into use. Damage due to failing to follow the instructions will render the warranty to lapse.

### Caution

- The Extender may only be used by persons aged 12 and older. People with reduced physical, sensory or mental abilities or a lack of experience and knowledge must be supervised when using the device.
- Special care is required when used by or near children. The Extender is not to be used as a toy for or by children. Children are not to clean or maintain the Extender.
- The Extender has moving parts that can pinch your hair, clothing and fingers. Be careful when using the device.
- Do not use the Extender for any purpose other than putting on therapeutic elastic stockings (TES) together with the powered HelpSoq On device.
- Not all types of TES stockings, nor all sizes and lengths are suitable for use with the Extender. Some very tight, narrow or long stockings can damage your Extender. On our website [helpsoq.nl/kousen](http://helpsoq.nl/kousen), you will find an up-to-date overview of stockings suitable for use with the Extender.
- Maintenance: the Extender can be cleaned using a damp microfibre cloth. If any liquid is accidentally spilled on or in the device, wipe it off using a microfibre cloth.
- The Extender meets all required safety standards. Repairs may only be carried out by authorised HelpSoq repair personnel. Inadequate, unqualified repair work may cause damage to the Extender.
- The Extender is designed for everyday use, to put on the most common TES stockings (see list on [helpsoq.com/kousen](http://helpsoq.com/kousen)) together with the electrical HelpSoq On device.

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## Instructions for use: how to use the Extender

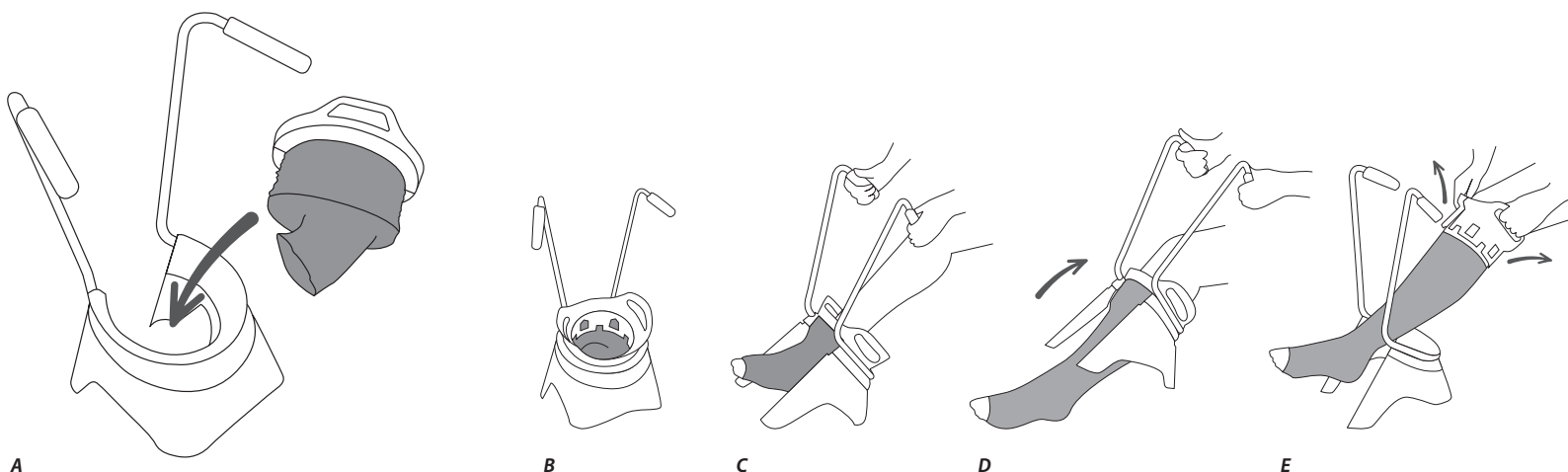
After rolling the stocking onto the ring, follow these steps using the the electric HelpSoq On device.

1. Place the Extender next to the HelpSoq on the table, or another convenient place within reach.
2. If the ring with the rolled-up TES stocking is still on the HelpSoq device, place your thumbs on top of the device and use your fingertips to lift the ring with the stocking off the HelpSoq.
3. Your fingertips prevent the stocking from slipping off the ring.
4. Look into the ring to see on which side the heel's stocking fabric sits. The opening for the toes (or in some stockings: closed toes) is on the other side.
5. Move the ring with the rolled-up stocking towards the Extender (fig. A) and let the ring with the stocking on it slide into the Extender. This way, the (open) toes must end up on the open side of the Extender.
6. The Extender now clamps the ring with the stocking, preventing the stocking from slipping off (fig. B).
7. Now take the Extender by the handles and place the device on the floor, next to your feet. The handles are now facing you, and the open side of the Extender is facing away from you. The ring with the stocking is fixed in the circle.
8. Now carefully step your foot into the stocking. By moving the handles (forward, backward, left or right) you can guide this entry somewhat. Make sure that the heel of your foot aligns nicely with the heel of the stocking. You can tell once it is in the right place (fig. C).
9. Now that the foot is sitting properly in the stocking, you can lift the handles as high as you wish or can (fig. D). The stocking will unwind, up your leg.
10. Once the stocking with ring is high enough, place the Extender back on the floor. You can let go of it now. Now your hands are free to grab the ring and pull it all the way up.
11. The stocking now unwinds completely up your leg, until there is no stocking left on the ring (fig. E). At this point, break open the ring and remove the two parts from your leg. Now click these two parts back together again and place them on the HelpSoq for the other stocking.
12. This step-by-step procedure allows you to put on the stocking without having to bend down.
13. Repeat all the steps in the HelpSoq user instructions for putting on the second stocking.

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## Troubleshooting

### **The Extender feels 'loose', as if something is not attached.**

The Extender should indeed feel wobbly and flexible. The device is designed to move with you and the stocking. So the sensation of something being detached is completely normal. Rest assured, the device is otherwise sturdy.

### **One of the handlebars of the Extender comes off the device.**

The brackets with handles of the Extenders are firmly attached to the device. In the unlikely event of one of the brackets coming off, it can be easily clicked back into the plastic guide of the device. If this continues to happen, please contact us.

### **Can I use the Extender to take off the stocking as well?**

Unfortunately, the Extender is only suitable for putting on your stockings. Note that other aids are available for taking off the stockings.

### **What is the best way to clean the Extender?**

The device can be cleaned using a damp microfibre cloth. If any liquid is accidentally spilled on or in the device, wipe it off using a microfibre cloth.

*If none of the above provides a solution, please contact our service department. Our customer service department can be reached during office hours at +31 800-3377337 (NL) and of course also by email: [info@helpsoq.com](mailto:info@helpsoq.com)*

## Contact our service department

For all other questions or correspondence, please contact our service centre at:

 HelpSoq Service Centre  
Hondiusstraat 28C, 6827DE Arnhem (NL)  
Tel. +31 800 3377 337 (Netherlands)  
Tel. +31 85 400 7744 (International)  
Online: [helpsoq.com/support](https://helpsoq.com/support)

Please follow our blogs at [helpsoq.com/news](https://helpsoq.com/news) to stay up to date with new insights and experiences.

This manual has been written with care by HelpSoq B.V. HelpSoq is not liable for any damages caused by inadequacies or errors in this manual. It is not an addition to the general terms and conditions and no rights can be derived from it. The complete HelpSoq service and guarantee conditions that apply to your device are registered at the Chamber of Commerce in the Netherlands. The latest version of these guarantee-conditions can be requested via our service organisation. The conditions can also be downloaded directly from [helpsoq.com/support](https://helpsoq.com/support).

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